Coating Free Underlap Area

Coating Free Underlap Area

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DIRECTIONS FOR USE: This product cleans, disinfects and deodorizes hard, nonporous medical surfaces in one step with no rinsing required. When used as directed, this product contains 1.40% (14,000 ppm) of active disinfectant making it highly effective against a broad spectrum of bacteria, viruses and fungi on hard, nonporous surfaces. This product is a one-step hospital-use disinfectant cleaner and deodorant designed for general cleaning and disinfecting hard, nonporous inanimate surfaces. Not for cleaning or sanitizing skin. Do not use as a diaper wipe or for personal cleansing. For surfaces and/or objects that may come into direct contact with food, a rinse with potable water is recommended. Do not use on glassware, utensils, or dishes. For surfaces and/or objects that may come into direct contact with children at the mouthing stage of development, a rinse with potable water is recommended.

To Open Package: Open lid and pull out wipe from centre of roll. Next sheet pops up automatically. Close lid to retain moisture.

To Clean and Disinfect Hard, Nonporous Surfaces: Wipe surface until completely wet. Allow this product to remain wet for contact time listed on the chart. Let air dry. For visibly soiled areas, a pre-cleaning is required.

Special Instructions for Use Against Human Immunodeficiency Virus (HIV-1), Hepatitis B Virus (HBV), and Hepatitis C Virus (HCV): This product kills HIV-1, HBV and HCV on pre-cleaned environmental surfaces/objects previously soiled with blood/body fluids in health care settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids. **Personal Protection:** Personnel that clean items soiled with blood or body fluids must wear appropriate personal protective equipment (i.e., barrier protection), such as disposable gloves, gowns, masks or eye coverings.

Cleaning: Target surfaces must be cleaned prior to disinfection, as the presence of heavy soil on a surface may reduce the intended disinfectant efficacy of the product. **Infectious materials disposal:** Any materials used in the cleaning process that may contain blood or body fluids are to be disposed of immediately in accordance with local regulations for infectious materials disposal.

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CAUTION: IRRITANT. LIQUID MAY IRRITATE EYES. Do not get in eyes or on clothing. For sensitive skin or prolonged use, wear rubber gloves. Avoid contact with food. **KEEP OUT OF REACH OF CHILDREN AND PETS. FIRST AID TREATMENT:** Contains Hydrogen Peroxide. If in eyes, rinse with water for 15 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye. Have the product container or label with you when calling a Poison Control Centre or doctor, or going for treatment.

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STORAGE AND DISPOSAL: Store this product in a cool dry area to avoid deterioration. Dispose of wipe in trash after use. Do not flush. Nonrefillable container. Do not reuse or refill empty container. Recycle empty container or discard in trash.

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based on aseptic technology

Hydrogen Peroxide Cleaner Disinfectant Wipes

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DIN 02406225

- Kills TB in 1 minute
- Kills 56 Pathogens
- One-Step Disinfectant Cleaner
- Cleans, Disinfects, Deodorizes
- Compatible with Equipment Surfaces

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KEEP OUT OF REACH OF CHILDREN AND PETS

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ARTWORK & IMAGES RIGHTS	RIGHTS: ● HAVE ● LIMITED ● NO ● ELECTRO
(File Name or Description) and Source	
●	Stopwatch illustration created by Mosaic (Modified by Don Demagajcs)
●	Not a BabyWipe artwork created under Anna Rodrigues (Chu) in 2008. PG#20211, WO#10758
●	Aseptix logo
●	Clorox logo (Source: CBX)
●	CloroxPro logo (Source: CBX)
●	Do Not Flush illustration. (Source: AISE)
○	File info here

Notes:

09/01/20	1	AS	C&S
DATE	REV	TECH	LABEL TYPE

BLEED

CUT TOLERANCE

FINAL TRIM

NO TEXT AREA

We power great packaging

2009 West Avenue South
La Crosse, WI 54601
(608)788-5800

Process Cyan	Process Magenta	Process Yellow	Process Black	Orange	Green	Blue																			
FILENAME	ROUND	DATE	03/30/22 - 11:04 AM	PRINTER	Inland																				
194702.368_PPDHCHydroPeroxCDWipes95ct_2022_CA_i01	01	METHOD	Lithography	PRINT QUE	Inland_Litho-7C_C15-Paper-OGB																				
1	2	3	5	10	25	40	50	60	75	90	95	100	1	2	3	5	10	25	40	50	60	75	90	95	100
1	2	3	5	10	25	40	50	60	75	90	95	100	1	2	3	5	10	25	40	50	60	75	90	95	100
1	2	3	5	10	25	40	50	60	75	90	95	100	1	2	3	5	10	25	40	50	60	75	90	95	100
1	2	3	5	10	25	40	50	60	75	90	95	100	1	2	3	5	10	25	40	50	60	75	90	95	100

CONTRACT PROOF FOR COLOR & CONTENT

CLIENT
Clorox

JOB NUMBER
28724_1078401

1221 BROADWAY
OAKLAND CA 94612
510-271-7863

OPERATOR
John McGuire

APPLICATION
AI CS2020 v24.0.3

OGB

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Authorized user of registered trademark / Utilisateur autorisé de la marque déposée Clorox Healthcare, owned by / qui est la propriété de The Clorox Company, Oakland, CA 94612.