



Disinfection Best Practices Using Clorox® Reusable Wipe Containers

Product Information

Clorox Healthcare® Reusable Wipe Containers

Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes



Bucket: 2/185 ct., Item No. 01458
Refill: 2/185 ct., Item No. 01459

Clorox Healthcare® Bleach Germicidal Wipes



Bucket: 2/110 ct., Item No. 01309
Refill: 2/110 ct., Item No. 01310

Clorox Healthcare® VersaSure Alcohol-Free Cleaner Disinfectant Wipes



Bucket: 2/110CT., Item No. 31759
Refill: 2/110 ct., Item No. 31761

Directions for Use



1. REMOVE empty refill pouch from container.



2. WIPE inside and outside of container with fresh wipe until thoroughly wet.



3. To DISINFECT, keep container wet for required contact time.



4. REPLACE refill pouch once tub is dry. Thread and place lid on tightly.

Key Points

Retain bucket and lid for reuse. Refill pouches may be ordered separately to save money and the environment. Refill pouch should be **cut open** at top indication and **sides rolled down** before being placed in the empty bucket. Pull first wipe from center and feed through teeth of lid to pull one wipe at a time and remove excess liquid in the process. Replace lid securely on container.

Expiration date is printed on the refill package not the reusable tub.

For more information, contact your CloroxPro® sales representative.
Call: 1-866-789-4973
Web: www.cloroxpro.ca

© 2021 Clorox Professional Products Company,
150 Biscayne Crescent, Brampton, ON L6W 4V3

