HEALTHCARE™

Spore Defense™

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DIRECTIONS FOR USE: This product is not for use on critical and semi-critical medical device surfaces. To disinfect, Clean and Deodorize Hard, Nonporous Surfaces: Spray surface until thoroughly wet. Allow the surface to remain wet for contact times listed in the table.

Instructions for Clostridium difficile: This product is intended for use to kill or inactivate spores of C. difficile in settings where they are expected to be encountered, such as settings where contamination by fecal matter is likely. **Personal Protection:** Personnel that clean items potentially soiled with spores of C. difficile must wear appropriate personal protective equipment, such as disposable gloves, gowns, masks or eye coverings. **Cleaning Procedure:** Target surfaces must be cleaned prior to disinfection, as the presence of heavy soil such as fecal matter on a surface may reduce the intended disinfectant efficacy of the product. **Infectious Materials Disposal:** Any materials used in the cleaning process that may contain feces/waste are to be disposed of immediately in accordance with local regulations for infectious materials disposal. **Instructions for bloodborne viral pathogens (HBV, HIV, HCV):** This product is intended for use against the bloodborne pathogens in settings where the viruses would be expected to be encountered, such as settings where contamination by blood or body fluid is likely. **Personal protection:** Personnel that clean items soiled with blood or body fluids must wear appropriate personal protective equipment (i.e., barrier protection), such as disposable gloves, gowns, masks or eye coverings. **Cleaning:** Target surfaces must be cleaned prior to disinfection, as the presence of heavy soil on a surface may reduce the intended disinfectant efficacy of the product. **Infectious materials disposal:** Any materials used in the cleaning process that may contain blood or body fluids are to be disposed of immediately in accordance with local regulations for infectious materials disposal.

DISINFECTS 42 OUT OF 47 PATHOGENS IN JUST 1 MINUTE

Bacteria: 1 minute contact time: *Styptococcus pyogenes*, *Vancomycin Resistant Staphylococcus aureus* -VISA, *Legionella pneumophila*, *Listeria monocytogenes*, *Serratia marcescens*, *Shigella dysenteriae*, *Staphylococcus epidermidis* - Coagulase Negative (CONS), *Salmonella enterica*, Carbapenem Resistant *Klebsiella pneumoniae*, Extended-Spectrum beta-lactamase (ESBL) producing *Escherichia coli* Multi-Drug Resistant, *Enterococcus faecium*, Multi-Drug Resistant *Styptococcus aureus*, Methicillin Resistant *Staphylococcus aureus* - MRSA, New Delhi metallo-beta-lactamase 1 (NDM-1) producing *Enterobacter cloacae*, *Pseudomonas aeruginosa*, *Staphylococcus aureus* (Staph), *Aerobacter baumannii*, *Proteus mirabilis*, Vancomycin Resistant *Enterococcus faecalis* -VRE, *Enterococcus faecalis*, *Enterobacter aerogenes*, *Escherichia coli* O157:H7. 1 minute 30 seconds contact time: Multi-Drug Resistant (MDR) *Styptococcus pneumoniae*.

Spore forming bacterium: 5 minutes contact time: *Clostridium difficile* (spore form).

Viruses Non-enveloped: 1 minute contact time: Enterovirus EV-06A, Human Rotavirus, Coxsackievirus type B3, Poliovirus Type 1, Human Adenovirus type 5, Rhinovirus type 37, Murine Norovirus, Hepatitis A virus.

Viruses enveloped: 1 minute contact time: Measles Virus, Mumps Virus, Avian Influenza A (H5N1) virus, Human Coronavirus (ja cause of the common cold), Influenza B Virus, Respiratory syncytial virus (RSV), Herpes simplex virus type 1, Bovine Viral Diarrhea Virus (Surrogate Virus for Human Hepatitis C Virus), Human Immunodeficiency virus type 1, Influenza A virus (H1N1), Duck Hepatitis B Virus (Human Hepatitis B Surrogate).

Fungi: 1 minute contact time: *Candida glabrata* 1 min 30 sec contact time: *Candida albicans*. 2 minutes contact time: *Trichophyton interdigitale*. 5 minutes contact time: *Aspergillus brasiliensis*.

CAUTION: IRRITANT. MAY IRRITATE EYES. Do not get in eyes, skin or clothing. Do not breathe fumes. KEEP OUT OF REACH OF CHILDREN AND PETS. Wash thoroughly with soap and water after handling. **FIRST AID:** Contains sodium hypochlorite. If swallowed, call a Poison Control Centre or doctor immediately. Do not induce vomiting. If in eyes, rinse with water for 15 minutes. If on skin, rinse well with water.

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AUTHORIZED USER OF REGISTERED TRADEMARK / UTILISATEUR AUTORISÉ DE LA MARQUE DÉPOSÉE CLOROX HEALTHCARE, OWNED BY / QUI EST LA PROPRIÉTÉ DE THE CLOROX COMPANY, OAKLAND, CA 94612.
Imprimé by / imprimé par: THE CLOROX COMPANY OF CANADA, LTD., 150 Niagara Cres., Brampton, Ontario L6W 4V3